ArtFun+

INSTRUCTIONS FOR USE

Version 0 – January 23, 2023

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<https://www.imageens.com/>

CAREFULLY READ ALL INSTRUCTIONS PRIOR TO USE. OBSERVE ALL WARNINGS AND PRECAUTIONS NOTED THROUGHOUT THESE INSTRUCTIONS. FAILURE TO DO SO MAY RESULT IN COMPLICATIONS.

Document reference: REC_IFU_20230123_1

The present Instruction for use is available on Imageens' website at:
<https://www.imageens.com/ifu-artfun>

This document contains confidential information that belongs to IMAGEENS. No part of this document may be transmitted, reproduced, used, or disclosed outside of the receiving organization without the express written consent of IMAGEENS.

TABLE OF CONTENT

GENERAL WARNINGS.....	4
NOTES	4
1 GENERAL	5
1.1 Biomarker & Stiffness quantification	5
1.2 ArtFun + Description	6
1.2.1 Intended use	6
1.2.2 Principle of operations.....	6
1.2.3 Contraindication, precautions, and product warnings.....	8
1.2.4 Manufacturer's information	9
2 Technical Requirements	9
3 Getting started with ArtFun+	11
3.1 ArtFun+ awareness videos	11
3.2 Check Software Integrity.....	11
3.3 Installation.....	12
3.4 Import series with Osirix or Horos	12
3.5 Start the plugin: Get a license key	13
4 ArtFun+ Layout	14
4.1 Viewer	14
4.2 Metadata Panel	15
4.2.1 Patient metadata	15
4.2.2 Study and series identifiers	17
4.3 Viewers controls.....	18
4.4 Time frame / anatomical position slider	18
4.5 Workflow controls.....	19
4.6 Results panel	19
5 AAD CALCULATION	20
5.1 Aortic section segmentation	20
5.1.1 Automated segmentation.....	20
5.1.2 Adjust contour	22
5.1.3 Edit contour	23
5.2 Assignment of a label	23
5.3 Computation of indicators including AAD.....	23
6 PWV CALCULATION.....	24
6.1 Aorta centerline	24
6.2 Set aorta points.....	24
6.2.1 Edit aorta points	25
6.2.2 Aortic length & 3D view.....	25
6.3 Transit time calculation.....	26
6.4 Computation of indicators including PWV	26

7	ARTFUN+ RESULTS	27
7.1	Report.....	27
7.1.1	Flow visualization and correction	27
7.1.2	Background velocity correction (cine PC sequence only).....	28
7.1.3	Cine (2D+t) SSFP.....	29
7.2	CSV Export.....	30
8	ARTFUN+ POP-UPS.....	31
8.1	Informative pop-ups.....	31
8.2	Action pop-ups	32
9	CLOSE ARTFUN+	32
10	ARTFUN+ UPDATES	32
11	UNINSTALL ARTFUN+	33
12	ARTFUN+ PERFORMANCES	33
13	USER DOCUMENTATION & SUPPORT	35
13.1	HELP MENU	35
13.2	SUPPORT	35
14	SYMBOLS.....	35
15	REFERENCES.....	36

GENERAL WARNINGS

- This manual gives important information about the use and safety of this device. Deviating from operating procedures, misuse or misapplication of the device, or ignoring specifications and recommendations could result in increased risk of harm to users and patients.
- Users are expected to be qualified healthcare professionals knowledgeable about medical procedures and patient care. Before attempting to use this device for clinical applications, the user must read and understand the contents of the Instructions For Use. Inadequate knowledge could result in increased risk of harm to users and patients.
- ArtFun+ results are not intended to be the sole means of prediction, prognosis or prevention. It is offered to healthcare professionals on an advisory basis only in conjunction with the patient's clinical history, symptoms, and other diagnostic tests, as well as the clinician's professional judgment and other prediction, prognosis, prevention and/or diagnosis information.
- Any serious incident that has occurred in relation to the device should be reported to the IMAGEENS and the competent authority of the Member State in which the user and/or patient is established.

NOTES

- This manual may contain screenshots. Any screenshots are provided for reference only.
- IMAGEENS is responsible for the effects on safety and performance only if the device is used in accordance with the instructions for use.
- No residual risks were identified for Artfun+ use.

1 GENERAL

1.1 BIOMARKER & STIFFNESS QUANTIFICATION

Changes in aortic stiffness have a high physiopathological relevance as they can lead to increases in the aortic pulse pressure [1,2] and the cardiac pressure afterload, which can cause left ventricular hypertrophy [3]. Arterial stiffness is recognized as a major risk factor in coronary heart disease [4,5], and is considered as an independent predictor of cardiovascular mortality [6,7,8,9,10]. It is therefore increasingly used in clinical practice [11].

Distensibility and pulse wave velocity (PWV) are commonly used to characterize the arterial stiffness [12,13,14,15,16]:

- The distensibility describes the ability of the artery to expand during systole and is defined as the relative change in the cross-sectional area of the artery (strain) divided by the local pulse pressure.
- The PWV is the propagation speed of the pulse wave along the artery and is calculated as the ratio between the distance separating two locations and the transit time needed for the wave to cover this distance.

Distensibility: local index for stiffness

Combining the estimation of the deformability and the pulse pressure, a local index of distensibility in the arterial section can be derived.

The distensibility is computed as:

$$D = \frac{\Delta A}{\Delta P} = \frac{\text{Strain}}{\text{Brachial/Carotid PP}}$$

Pulse Wave Velocity: regional stiffness index

Combining the estimation of the transit time and the estimation of the length of the aortic arch, a regional index of Pulse Wave Velocity in the aortic arch can be derived.

PWV is computed as:

$$PWV = \frac{\Delta L}{\Delta T} = \frac{\text{Aortic length}}{\text{Transit time}}$$

1.2 ARTFUN + DESCRIPTION

1.2.1 INTENDED USE

ArtFun+ is intended to be used for viewing, post-processing, and quantitatively evaluating Magnetic Resonance Images (MRIs) of the following type: aortic cine 2D+t (sequence of 2D images over time) Steady-State Free Precession (SSFP), aortic cine 2D+t Phase Contrast (PC) and thoracic 3D SSFP in a DICOM standard format.

It allows the calculation of MRI Ascending Aorta Distensibility (AAD) and aortic arch MRI Pulse Wave Velocity (PWV).

Artfun+ results are not intended to be the sole means of **therapeutic or diagnostic decision**. It is offered to healthcare professionals on an advisory basis only in conjunction with the patient's clinical history, symptoms, and other diagnostic tests, as well as the clinician's professional judgment and other prediction, prognosis, prevention and/or diagnosis information.

1.2.2 PRINCIPLE OF OPERATIONS

ArtFun+ software allows the user:

1. to load and display following MRI images
 - Thoracic 3D *Steady-State Free Precession* (SSFP) acquisitions
 - Cine 2D+t SSFP acquisitions
 - Cine 2D+t *Phase-Contrast* (PC) acquisitions
2. to automatically segment (i.e. delineate) the aortic cross sections in Cine images (semi-automatic and pure manual delineation are available for difficult cases)
3. to manually position the aortic centerline in thoracic 3D images
4. to compute from these *Regions Of Interest* (ROIs), clinical indices

The results of the Ascending Aortic Distensibility (AAD) and Pulse Wave Velocity (PWV) biomarkers calculation are available on Artfun+ interface (see figures 2 & 4) and can be downloaded as .csv file.

Artfun+ software works in the following sequence:

- Installation of Artfun + with a valid license key
- Selection of the images to analyze from Osirix/Horos which launches the Artfun+ plugin automatically

For Distensibility calculation, the following steps shall be followed:

- Drawing of a bounding box that will allow for the aortic section detection on 2D+t sequence
- Launching of initial segmentation & segmentation computation
- Assignment of a label (color) to the segmentation

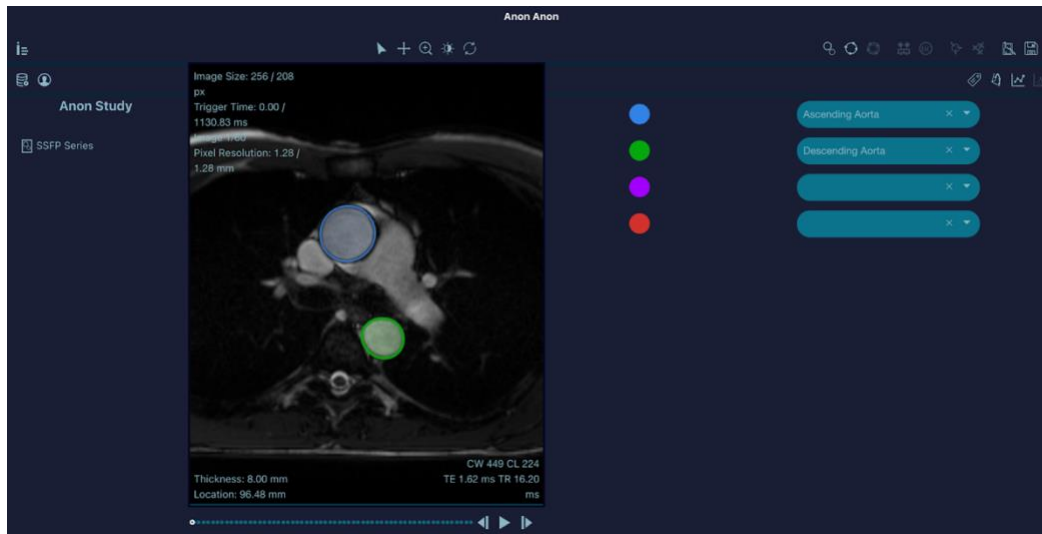
Figure 1 - Arterial segmentation of the ascending (blue ROI) and descending aorta (green ROI)

- Input of patient's pulse pressure
- Computation of indicators that are then displayed on the interface

Figure 2 - Distensibility biomarker calculation

For PWV calculation, the following steps shall be followed:

- Same process as for distensibility (as described above) for the two aortic sections (ascending & descending) to obtain the transit time (PC sequence only)
- Positioning of the aorta centerline on 3D sequence (see figure below)

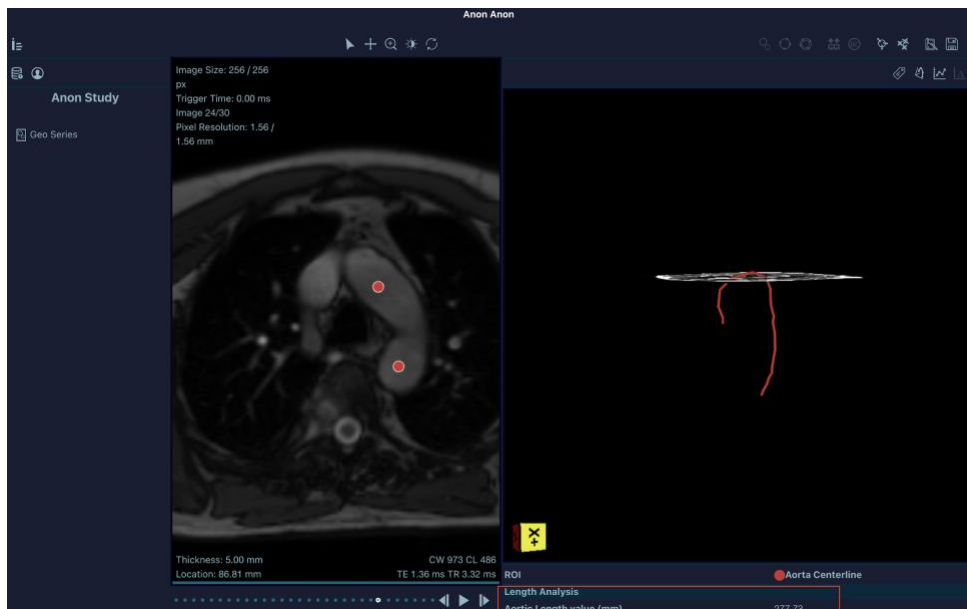
Figure 3 - Aorta centerline

- Input of patient's length of the arch of the aorta
- Computation of the Pulse Wave Velocity that is displayed on the interface

Figure 4 - PWV biomarker calculation

Once indicators are calculated the user can export the results in .csv format.

1.2.3 CONTRAINDICATION, PRECAUTIONS, AND PRODUCT WARNINGS

ArtFun+ is not indicated for patients with overt cardiovascular diseases.

MRI images must be compliant with DICOM standard and DICOM fields must not have been modified other than for anonymizing patient-related information.

The following product warnings are disseminated throughout the present document. And identified thanks to the WARNING icon as shown below.

WARNING

- The user shall ensure that no artefacts are present in the image after loading that may alter the segmentation of the aorta.
- If a PC image is loaded, care must be taken to ensure that the number of velocity images in the PC image is the same as the number of morphological images.
- Pulse pressure “Central PP (mmHG)” and “Aortic distance for MRI PWV (mm)” fields are used for AAD and/or PWV calculation. It is the responsibility of the user to ensure those values are correct to ensure adequate results.
- It is the responsibility of the user to enter a correct value for the Pulse pressure field “Central PP (mmHG)” to ensure AAD is correctly calculated by ArtFun+. Pulse pressure value order of magnitude shall be up to 200mmHG.
- It is the responsibility of the user to enter a correct value for the Aortic distance for MRI PWV to ensure PWV is correctly calculated by ArtFun+. The length of the aortic arch order of magnitude shall be in hundreds of mm.
- For the aorta centerline, only one point should be positioned per artery section per image (maximum 2 points by image). The 3D aorta will be computed even if you don't place all the points on all the slices or if you put more points than expected. It remains your sole responsibility to ensure the results computed are as precise as possible.
- The csv file used for export must be closed and not used by any other applications during the use of ArtFun+ to prevent corrupting the file.
- Make sure you save your work before closing the program. Any unsaved data will be lost when you close the program.

1.2.4 MANUFACTURER'S INFORMATION

ArtFun+ is a class IIa medical device manufactured by IMAGEENS. Below is the different information related to the manufacturer of this device:

Manufacturer Legal Address	42 Rue de Maubeuge 75009 Paris
Company's main office	1 Rue de l'Averse, 94000 Créteil
E-mail address	hello@imageens.com
Internet Website	https://www.imageens.com/

2 TECHNICAL REQUIREMENTS

In order to guarantee the proper functioning of the device, it is necessary to ensure that the computer operating ArtFun+ meets the criteria listed below:

Criteria	Requirement
Computer Processor	Dual Core Intel i5 or higher of 7th-gen/Kaby Lake architecture or higher OR Apple M1
Computer Memory	8 Go de RAM or higher
Computer OS	MacOS BigSur
Application previously installed	Osirix v11 or higher OR Horos v3.3 or higher

Table 1 – Computer Requirements

An internet connection of 100 Mb or more is also required when first activating the product license and at each start for license validation.

It is also necessary to check that the DICOM file used for each analysis meets the criteria shown in the table below:

Criteria	Requirement
Brand of the MRI used to generate the data	General Electric or Siemens
Image Resolution	Between 192x192 and 256x256 pixels
Temporal resolution	Lower than 30ms (i.e. 40 real phases or higher for a typical cardiac cycle/average heart rate)
Slice thickness	Between 5 and 8 millimeters
Pixel Resolution	Between 1 and 3 millimeters
Magnetic field strength	1,5 Tesla

Table 2 – DICOM Requirements

ArtFun+’s lifetime: Min. 1 year. This value is based on ArtFun+ MacOS compatibility. It can be extended depending on the technology evolution.

CYBERSECURITY Requirements

The following good practices are recommended to ensure protection, integrity and security of Artfun+ product and its data.

Implement the following IT minimum requirements

- Firewall
- Antivirus & malware protection on your computer

Do not use ArtFun+ when you are connected to an open network which is less secured.

ArtFun+ does not store any data, please ensure your results are extracted in .csv format before closing the application. Once the application is closed no data can be retrieved.

If you have any doubt that your product has been altered, you can check the software integrity by performing the tasks described in section 3.2 Check software integrity of the present Instruction for use.

If your product has been altered or do not use it and please contact IMAGEENS support (see section 13 of this instruction). If needed IMAGEENS support will send you a new product to ensure integrity is preserved.

3 GETTING STARTED WITH ARTFUN+

3.1 ARTFUN+ AWARENESS VIDEOS

In addition to the present Instruction for use, you have at your disposal 14 videos to help you understand how to use ArtFun+.

The following topics are available in the videos:

TUTO A – Data loading

TUTO B – Patient Info

TUTO C – Image viewer

TUTO D – Arterial Segmentation

TUTO E – Arterial Segmentation Manual Correction

TUTO F – Arterial Segmentation Semi- Automatic Correction

TUTO G – Segmentation Analysis

TUTO H – PC Background noise correction

TUTO I – Transit time & PWV

TUTO J – Distensibility

TOTO K – Export

TUTO L – Aortic Centerline

TUTO M – Aortic Centerline correction

TUTO N – Centerline Analysis

Those tutorials are in .mov format and shall be read with Quicktime player.

3.2 CHECK SOFTWARE INTEGRITY

To be able to check the integrity of the software to be installed here is the procedure to follow:

1. Access your Mac's terminal using the "command "+"space" key combination
2. Type "terminal" and launch "Terminal.app"
3. Type "md5 /...", "/"... being the path to the installation file
4. Press enter and check that the value to the right of the "=" sign corresponds to the MD5 hash value indicated in the product release note.

In case these values are different, do not install the product and contact IMAGEENS.

3.3 INSTALLATION

ArtFun+ software was designed as a plugin for Osirix and Horos.

1. Ensure the previously mentioned version of Horos OR Osirix is installed.
2. Unzip and launch the installer file by double click.
3. Osirix / Horos will start and ask you to install the plugin. You might be asked for your system password by macOS to allow the installation.
4. Osirix or Horos will ask you to restart to complete the installation.
5. After a restart of Osirix/Horos, you will find the plugin in the menu "Plugins" ► "Image Filters".
6. In the Osirix/Horos 2D Viewer panel, go in the menu "Format" ► "Customize Toolbar and drag&drop ArtFun+ into the usual toolbar to create a shortcut to ArtFun+.

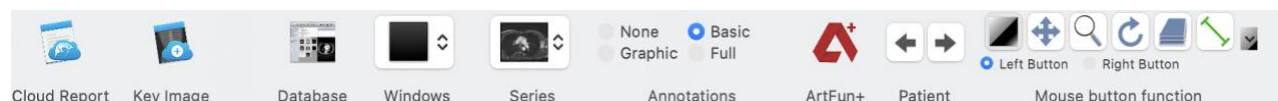
Figure 5 - Horos Tool bar

To verify the correct installation of ArtFun+, run ArtFun+. The plug-in should then start, and you should arrive at the window indicated in chapter 3.3.

If this is not the case, it may have been wrongly installed. You can check if Horos/Osirix find it in the "Plugins" ► "Plugins Manager" ► "Installed Plugins" tab.

For the uninstallation, open the "Plugin Manager" of Osirix or Horos ("Plugins" ► "Plugin Manager"). Select the plugin and uninstall it with the "Uninstall selected plugin" button.

3.4 IMPORT SERIES WITH OSIRIX OR HOROS

1. Import the cine (2D+t) PC or SSFP or 3D SSFP series that you want to examine to Osirix or Horos.
2. You can import DICOM files from CDs or other volumes or configure your local workstation as a DICOM network node to receive data directly from your local PACS network.
3. Open the series that you want to examine with Osirix or Horos by selecting the patient in the list and double click on the thumbnail. The series will appear in the Osirix/Horos 2D Viewer
4. Open ArtFun+: launch the plugin ArtFun+ via the plugin menu item or via the toolbar shortcut

WARNING

- The user shall ensure that no artefacts are present in the image after loading that may alter the segmentation of the aorta.
- If a PC image is loaded, care must be taken to ensure that the number of velocity images in the PC image is the same as the number of morphological images.

3.5 START THE PLUGIN: GET A LICENSE KEY

1. Start the plugin via the plugin menu item (Plugins ► Image Filters ► ArtFun+), from the 2D Osirix/Horos Viewer panel, or from your own shortcut on the toolbar.
 - If your computer is a Mac with an M1 processor and you don't have the Rosetta library installed, you might be prompted to install this library. This action is required for M1 Mac to be able to run ArtFun+.
 - If it is the first time you launch the plug-in you might be prompted, through a pop-up, to allow the plug-in to access your data.
2. The application prompt for the license key to activate the plugin
3. If you don't have one, you can request it by filling your email address in the first field and sending it through the "Request activation" button. We will receive your demand and contact you by email for the license key.
4. Once you have your license key, paste it in the second field and unlock the plugin through the "Activate license" button
5. The plugin is unlocked: you may have to click on it again in Osirix/Horos to get your first image.

The image shows a 'Software Activation' dialog box with a dark blue background. At the top, it says 'Software Activation' in white. Below that, it asks 'You don't have a key ?'. There is a text input field containing 'example@email.com'. Below the field is a blue button labeled 'Request activation'. Underneath the button, it says 'We will send you an email as soon as possible'. Then, it asks for the 'License Key' with a larger text input field. At the bottom is another blue button labeled 'Activate License'.

Figure 6 - ArtFun+ Activation

4 ARTFUN+ LAYOUT

Once the plugin is activated and you launch it from Osirix/Horos Toolbar, you will be directed to the ArtFun+ User Interface (UI).

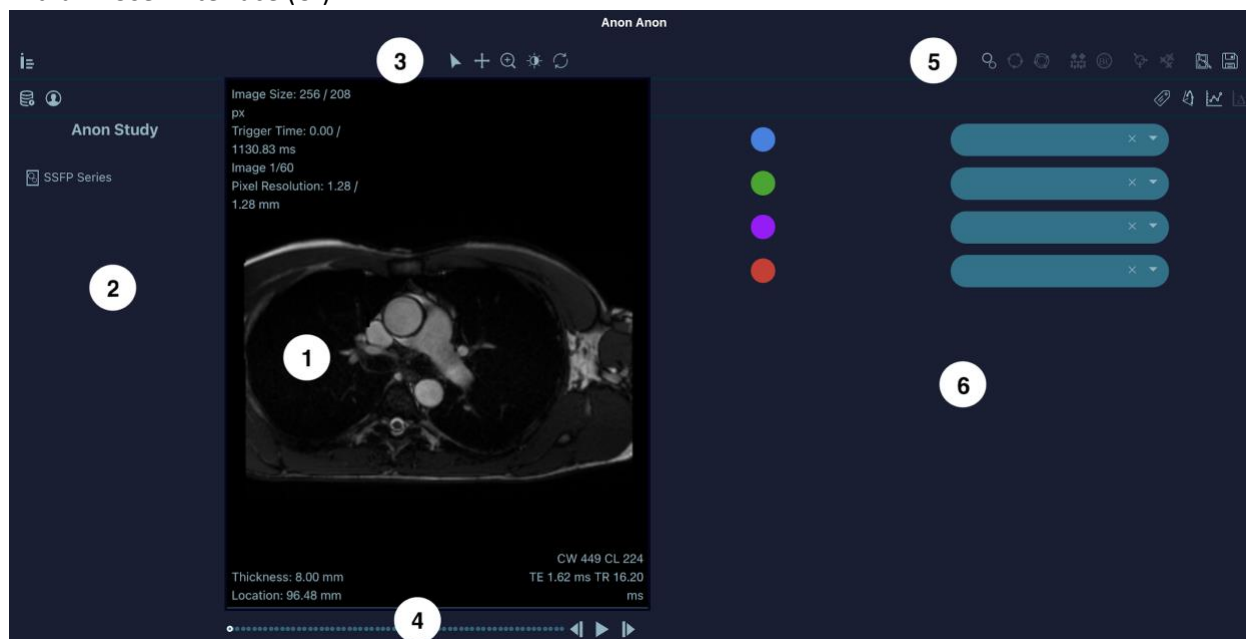
Figure 7 - ArtFun+ Viewer

- (1) Viewer: images are displayed here
- (2) Metadata panel: Find patient and study information
- (3) Viewer controls: Basic navigation tools
- (4) Time frame/anatomical position slider: Select which time point or anatomical position to display in viewer
- (5) Workflow controls: tools used for the analysis process
- (6) Results panel: ROI labeling, graph charts and clinical indices

In both (2) and (6) panels you can navigate in several sub panels by clicking on the different icon on the top.

4.1 VIEWER

The Viewer displays different images from a specific 2D+t or 3D series and provides the following information:

Image size	The size in pixels of the current image (columns / rows)
Trigger time	The trigger time (in ms) of the current image and the total sequence time
Image index	The index of the current image and the total number of images

Pixel size	The physical size comprised in a pixel (vertically / horizontally)
Slice thickness	The physical size imaged by the current image
Slice location	The physical relative position of the current image in the sequence
TE	The echo time used in the acquisition of the current image
TR	The repetition time used in the acquisition of the current image
CL & CW	The current color level and window (luminosity and contrast)

Table 3 - Viewer image information

4.2 METADATA PANEL

The Metadata panel shows information about the series that is currently loaded. The following icons are displayed:

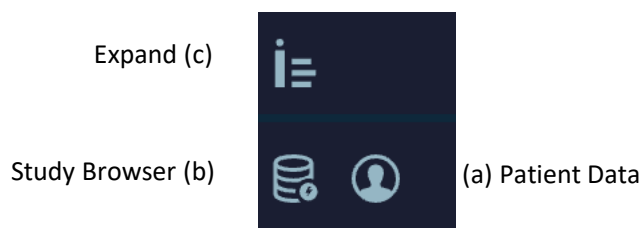
Figure 8 - Metadata Panel icons

There are 2 subpanels that show patient metadata by clicking on the icon **(a)** and study and series identifiers by clicking on the icon **(b)**.

You can expand or diminish the Metadata panel by clicking on the left panel icon **(c)**.

4.2.1 PATIENT METADATA

The following patient information is listed:

Name	Patient's name as provided by the PatientsName (0010,0010) DICOM field
ID	Patient's ID as provided by the PatientID (0010,0020) DICOM field
Birth Date	Patient's birth date as provided by the PatientsBirthDate (0010,0030) DICOM field
Sex	Patient's sex as provided by the PatientsSex (0010,0040) DICOM field
Age	Patient's age as provided by the PatientsAge (0010,1010) DICOM field, in years (y)
Weight	Patient's weight as provided by the PatientsWeight (0010,1030) DICOM field, in kilograms (kg)
Height	Patient's height as provided by the PatientsSize (0010,1020) DICOM field, in meters (m)

Heart Rate	Patient's heart rate as provided by the HeartRate (0018,1088) DICOM field, in beats per minute (bpm)
BMI	Patient's body mass index as provided by the PatientsBodyMassIndex (0010,1022) DICOM field, in kilograms per square meters (kg/m ²)
Body Surface Area	Patient's body surface area, in square meters (m ²)
Carotid SP	Patient's systolic blood pressure measured at the carotid, in millimeter of mercury (mmHg)
Carotid DP	Patient's diastolic blood pressure measured at the carotid, in millimeter of mercury (mmHg)
Carotid PP	Patient's pulse blood pressure calculated as Carotid SP – Carotid DP, in millimeter of mercury (mmHg)
Brachial SP	Patient's systolic blood pressure measured at the brachial artery, in millimeter of mercury (mmHg)
Brachial DP	Patient's diastolic blood pressure measured at the brachial artery, in millimeter of mercury (mmHg)
Brachial PP	Patient's pulse blood pressure calculated as Brachial SP – Brachial DP, in millimeter of mercury (mmHg)
Central SP	Patient's systolic blood pressure measured at the aorta, in millimeter of mercury (mmHg)
Central DP	Patient's diastolic blood pressure measured at the aorta, in millimeter of mercury (mmHg)
Central PP	Patient's pulse blood pressure calculated as Central SP – Central DP, in millimeter of mercury (mmHg)
Aortic Distance for MRI PWV	Aortic distance between the ascending and descending aorta section for PWV estimation (mm)
Tonometric PWV	Patient's pulse wave velocity measured by tonometry, in meter per second (m/s)
HDL cholesterol	Patient's high density cholesterol level, in millimole per liter (mmol/L)
LDL cholesterol	Patient's low density cholesterol level, in millimole per liter (mmol/L)
Total cholesterol	Patient's total cholesterol level, in millimole per liter (mmol/L)
Framingham score	Patient's Framingham risk score, in percent (%)
Smoker	Indicate if the patient is a smoker
Diabetes	Indicate if the patient has diabetes
High blood pressure	Indicate if the patient has hypertension
Angina	Indicate if the patient has angina
Medical history	Indicate relevant information about the patient's health

Table 4 – Patient Metadata

If the related information is available from the DICOM metadata of the loaded series, fields will be automatically filled. If not, fields are empty by default.

Except Name & ID, all other patient fields are editable by the user by clicking on the “empty” field as shown in figure 9 below to add relevant information for reporting.

Anon Anon	
Name	Anon Anon
ID	
Birth Date	empty
Sex	empty
Age	empty
Weight (Kg)	empty
Height (m)	empty
Heart Rate (bpm)	empty
BMI (Kg/m2)	empty
Body Surface Area (m2)	empty
Carotid SP (mmHg)	empty
Carotid PP (mmHg)	empty
Carotid DP (mmHg)	empty
Brachial SP (mmHg)	empty
Brachial DP (mmHg)	empty
Brachial PP (mmHg)	empty
Tonometric PWV (m/s)	empty
HDL cholesterol (mmol/L)	empty
LDL cholesterol (mmol/L)	empty
Total Cholesterol	empty
Framingham Score (%)	empty
Smokers	empty
Diabetes	empty
High Blood Pressure	empty
Angina	empty
Medical History	empty

Figure 9 - Patient Metadata edition

WARNING: Pulse pressure “Central PP (mmHG)” and “Aortic distance for MRI PWV (mm)” fields are used for AAD and/or PWV calculation. It is the responsibility of the user to ensure those values are correct to ensure adequate results.

4.2.2 STUDY AND SERIES IDENTIFIERS

The study name and series names that are currently loaded are shown.

Anon Study	
Aorte PC Series	
Aorte PC Series_P	

Figure 10 - Study identification

4.3 VIEWERS CONTROLS

The icons right above the viewer are basic navigation tools to control the display of images.

Icon	Name	Description	Shortcuts
	Default tool	Return to default behavior and cancel any pending action.	Left click + drag onto viewer to move the image. Mouse wheel up/down to zoom in/out. Shift + left click + drag up/down onto viewer to adjust contrast, and left/right to adjust luminosity
	Panning tool	Move the field of view	Left click + drag onto viewer to move the image
	Zoom tool	Change the zoom level	Left click + drag up/down onto viewer to zoom out/in
	Luminosity-contrast tool	Change the luminosity and contrast	Left click + drag up/down onto viewer to adjust contrast, and left/right to adjust luminosity
	Reset tool	Reset the viewpoint to initial position	None

Table 5 - Viewer icons

4.4 TIME FRAME / ANATOMICAL POSITION SLIDER

As show on Figure 7 below, the Time frame / anatomical position slider is the bottom bar just below the viewer that displays and controls which time point or anatomical position is selected (depending on the type of image loaded, 2D+t or 3D respectively).

You can review this sequence image by image using the “backward”/“forward” buttons or by direct click on one dot. The sequence can also be played dynamically in cine mode with the “play”/“stop” button.

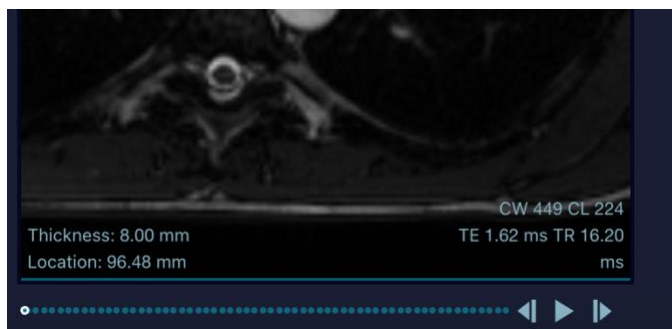
Figure 11 - Time Frame / Anatomical position slider

4.5 WORKFLOW CONTROLS

The workflow controls are used in the arterial analysis: segmentation, post-processing and reporting functions. The tools shown here are context sensitive such that they will be enabled or disabled depending on which type of MRI sequence is active. Below is a description of the tools shown:

Icon	Name	Description
	Vessel segmentation	(cine sequence only) Automatic spatio-temporal segmentation of the large vessels, acquired using cine (2D+t) SSFP or PC sequence, relying on our clinically validated algorithm.
	Adjust contour	(cine sequence only) Semi-auto correction of the ROI's contour using a manual template.
	Edit contour	(cine sequence only) Manual correction of the ROI's contour using a paintbrush-like tool.
	Show velocity layer	(cine PC sequence only) Switch view to the associated phase image displaying the velocities.
	Set background correction	(cine PC sequence only) Set a background velocity noise ROI to correct the velocity values.
	Set aorta points	(3D sequence only) Manual setting of aorta centerline points.
	Edit aorta points	(3D sequence only) Correction of the ROI's path by moving already set points.
	Report	Open/close the report tab to review all the results of the current session.
	Export CSV	Open the CSV export window to save the results locally on the computer (files are located on the desktop, in an "Imageens Report" folder)

Table 6 - Workflow controls icons

4.6 RESULTS PANEL

Four sub-panels are provided:

Icon	Name	Description
------	------	-------------

	ROI labels	Label up to 4 computed ROIs with a color and a name.
	Arterial graphs and indicators	Review the temporal curve of the arterial area during the cardiac cycle, and blood velocity or flow (for PC acquisition), and the main clinical indices automatically calculated on the curves.
	Flow delay estimation	(PC sequence only) Provide the transit time between 2 flow curves (mainly between the ascending and descending aorta)
	Aortic length & 3D view	Review the created centerline as well as the computed aortic length.

Table 7 - Sub-panels icons

5 AAD CALCULATION

For Ascending Aorta Distensibility (AAD) calculation, you should import Cine 2D+t SSFP acquisitions.

5.1 AORTIC SECTION SEGMENTATION

5.1.1 AUTOMATED SEGMENTATION

Once ArtFun+ displays your image, launch the automated segmentation by clicking on the “Vessel segmentation” icon from the workflow control panel (see section 4.5 above, Table 6). This tool is dedicated to the arterial section segmentation on cine sequence: Ascending aorta, descending aorta or any pseudo-circular section. The following steps shall be performed for arterial segmentation:

1. Draw a rectangle around the arterial section by left click + drag onto viewer

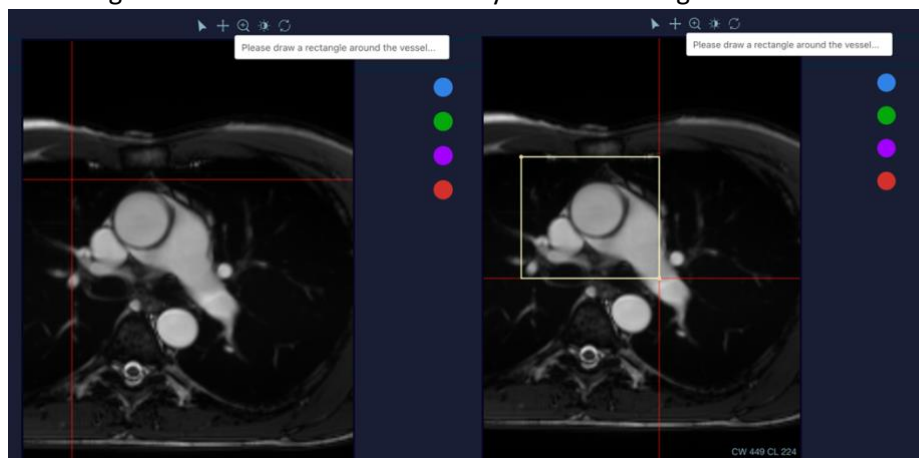
Figure 12 – Vessel Segmentation

2. An automatic circular initialization is proposed:

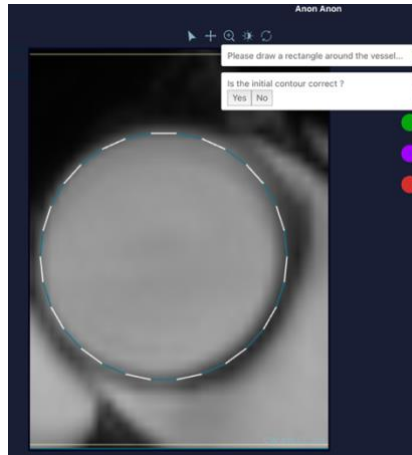
Figure 13 - Automatic circular initialization

3. Accept it if it is correct or reject it to do a manual initialization. To do so, click one point for the circle center, then one point for the circle ridge.

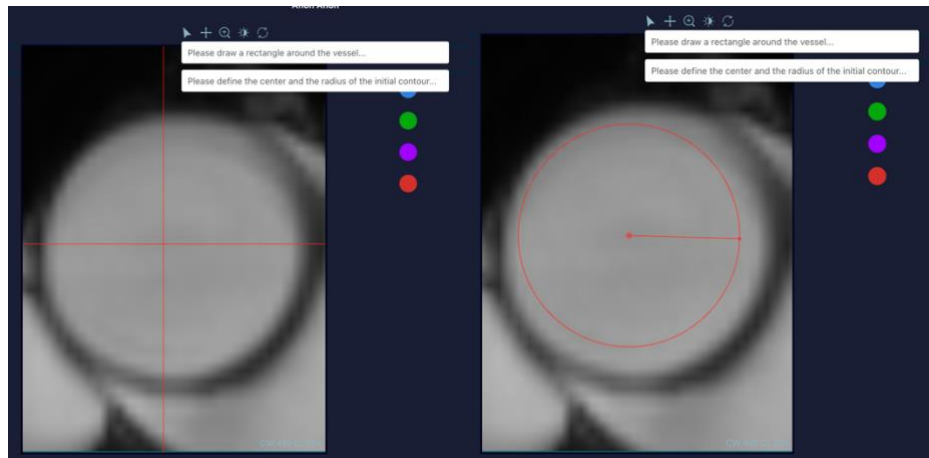
Figure 14 - Correction

4. The final segmentation is then automatically computed and displayed as an unlabeled ROI.

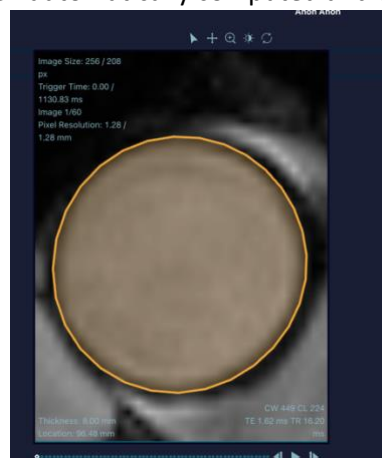
Figure 15 - Segmentation

5.1.2 ADJUST CONTOUR

If the computed contour is faulty, it can be semi-automatically corrected by clicking on the “Adjust contour” button from the workflow control (see section 4.5 above, Table 6).

1. Select the desired ROI before assigning it a label (Attention: if the Roi is labeled the functionality is deactivated.)
2. Click the Adjust Contour button
3. Identify the range of slices where the contour is faulty by selecting the first and the last images on the time frame / anatomical position slider. To confirm selection, click on the pop up “Select first/second slice” as shown on the figure below.

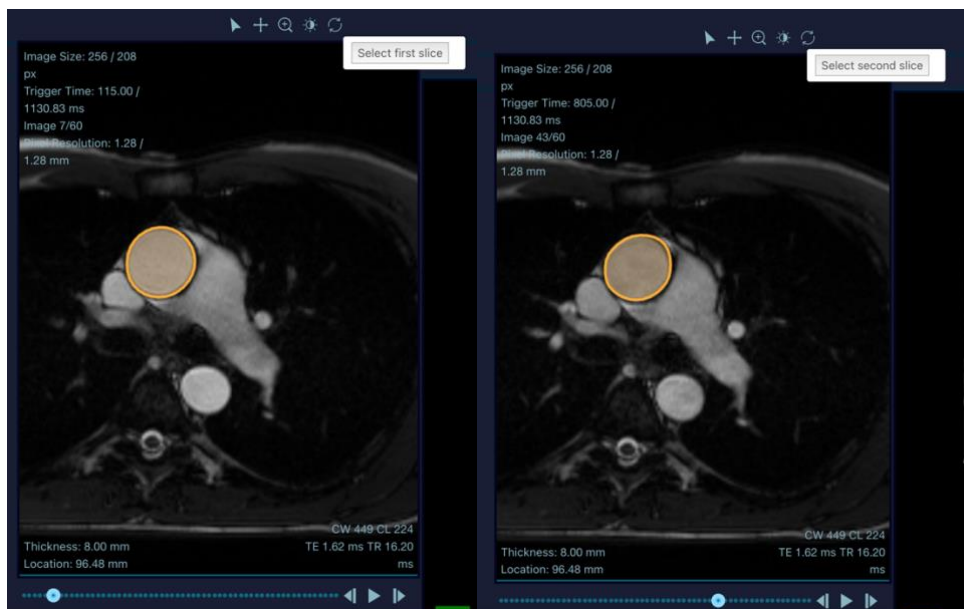
Figure 16 - Slice selection

4. Delineate the correct border of ROI on presented first slice and validate

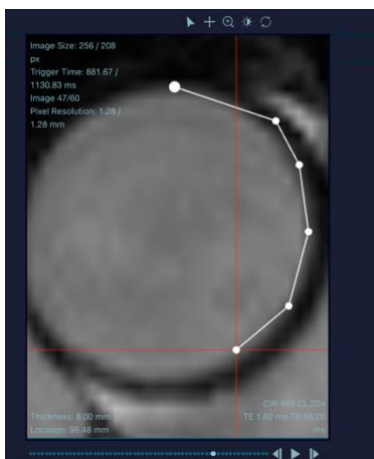
Figure 17 - ROI border delineation

5. Repeat for the presented last slice (only if different from first)

6. Automatic update of the contour is computed based on the manual template given in the two previous steps.

5.1.3 EDIT CONTOUR

If the computed contour is faulty, it can be manually corrected by clicking on the “Edit Contour” button from the workflow control (see section 4.5 above, Table 6).

1. Select the desired ROI
2. Click the Edit Contour button
3. Maintain left click onto the viewer to display the paintbrush tool. The farther from the contour the click, the larger will be the brush.
4. Move the contour by pushing it with the brush tool. To expand contour, click have to be inside contour, to shrink it on the outside
5. If the ROI is labeled, clinical indices are automatically recomputed from corrected contour on the left click release.

5.2 ASSIGNMENT OF A LABEL

Up to 4 Region Of Interest (ROI) can be analyzed concurrently. The following steps shall be done:

1. Select a ROI on the viewer.
2. Select a color disk to assign the color.
3. Select a name for the ROI among the proposed tags or write a custom one.

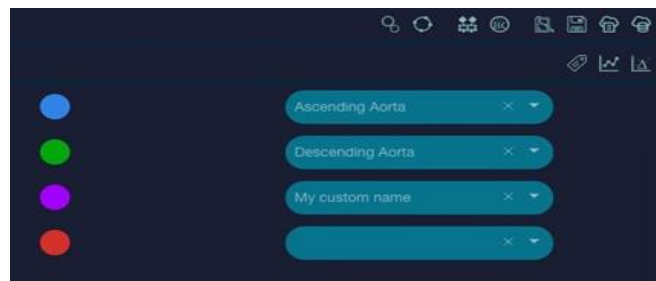
Figure 18 - ROI Labels

5.3 COMPUTATION OF INDICATORS INCLUDING AAD

Once the ROI has been labeled, the patient pulse pressure (“Central PP (mmHG)”) shall be included in the Patient Meta Data Panel (See above section 4.2.1, Table 4).

WARNING: It is the responsibility of the user to enter a correct value for the Pulse pressure field “Central PP (mmHG)” to ensure AAD is correctly calculated by ArtFun+. Pulse pressure value order of magnitude shall be up to 200mmHG.

Computation of indicators are then displayed on the Results Panel of the interface (See Figure 2). The complete list of computed indicators is provided in section 7.1 of the present instruction for use.

6 PWV CALCULATION

6.1 AORTA CENTERLINE

To define the aorta centerline, you must launch a 3D sequence.

6.2 SET AORTA POINTS

Once the 3D sequence is displayed, click on the “Set aorta points” button from the Workflow control panel (see section 4.5 above, Table 6). This tool, dedicated to the creation of an aortic centerline, enables you to select points on the image that represent the center of the aorta lumen.

To create the aorta centerline, you should place one point (left click) in the center of the aorta lumen on each slice. You can select each slice one by one thanks to the slider (See section 4.4 above).

If the image displays both ascending and descending aorta, you should place 2 points, one for each center.

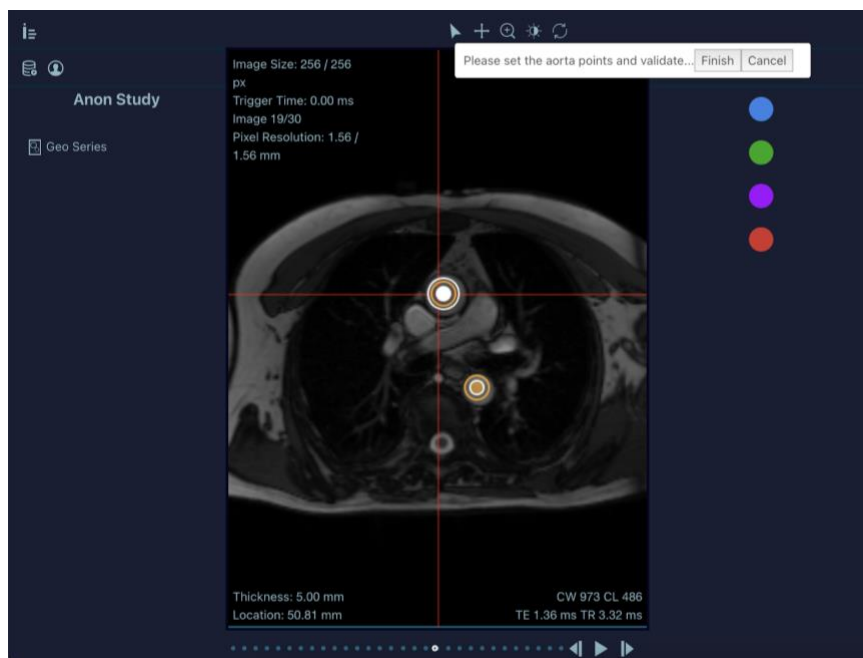
Figure 19 - Aorta center points setting

WARNING: For the aorta centerline, only one point should be positioned per artery section per image (maximum 2 points by image). The 3D aorta will be computed even if you don't place all the points on all the slices or if you put more points than expected. It remains your sole responsibility to ensure the results computed are as precise as possible.

To delete or modify an aorta point, see section 6.1.2 - Edit aorta point.

Once you have placed all the points, you shall validate by clicking on the “Finish” button. The final point set is displayed as an unlabeled ROI.

6.2.1 EDIT AORTA POINTS

If the centerline is faulty, it can be manually corrected by clicking on the “Edit Aorta points” button from the Workflow control panel (see section 4.5, Table 6).

1. Select the desired ROI
2. Click the Edit aorta points button
3. Drag & drop relevant points to their new location or delete them by pressing the backspace key
4. If the ROI is labeled, clinical indices are automatically recomputed from corrected path on the left click release.

6.2.2 AORTIC LENGTH & 3D VIEW

Once the aorta points have been validated, click on the button “Aorta length & 3D view” from the Result Panel (See section 4.6, Table 7) to display the 3D view that shows the created centerline over the anatomical.

Figure 20 – Aorta Centerline 3D view

On this view, you can:

- Rotate the view with a left click
- Zoom in/out with the mouse wheel or the right click
- Change the focal point with Shift + right click

The computed aortic length is displayed on the bottom part of the panel.

Indices	Units	Description
---------	-------	-------------

Aortic length	Millimeters (mm)	Total length of the centerline
---------------	------------------	--------------------------------

Table 8 - Aortic length units

6.3 TRANSIT TIME CALCULATION

Open PC serie & perform the segmentation (as described in section 5 above) for two aortic sections (Ascending & Descending) to obtain the transit time (Transit time will be available in the Results panel of the interface, in the sub panel Flow delay estimation under the “Ascent” field).

6.4 COMPUTATION OF INDICATORS INCLUDING PWV

Once the previous steps have been performed, the aortic arch length shall be included in the Patient Meta Data Panel, in the field “Aortic distance for MRI PWV (mm)” (See above section 4.2.1, Table 4).

WARNING: It is the responsibility of the user to enter a correct value for the Aortic distance for MRI PWV to ensure PWV is correctly calculated by ArtFun+. The length of the aortic arch order of magnitude shall be in hundreds of mm.

Computation of indicators is automatically performed by ArtFun+. To display the results, click on the “Flow delay estimation” sub panel from the result panel (See above section 4.6, Table 7) and then select the 2 curves by clicking on their respective colors disk.

Normalized curves can be displayed to get a better visualization of curve delay.

Figure 21 – Curve display

The interface then displayed both curves and the computed results including PWV (See above Figure 4).

The complete list of computed indicators is provided in section 7.1 of the present instruction for use.

7 ARFTUN+ RESULTS

7.1 REPORT

A reporting tab that resumes all the results of the current session is available by clicking on the ‘Report’ button from the Workflow panel of the interface (See above section 4.5, Table 6). The structure is as follow:

- General patient and study information
- Area measurements
- Flow measurements (PC sequence only)
- Transit Time measurement (PC sequence only)
- Value of biomarkers (if previously calculated)

7.1.1 FLOW VISUALIZATION AND CORRECTION

You can display the velocities on the viewer through the Show velocity layer icon (a), and correct it with the Background Correction tool (b). You can switch back to morphology image with the Show magnitude layer icon (c).

Figure 22 - Display velocity

Once you have performed the segmentation of the arterial section and labeled the ROI, the area, flow and velocity curves are automatically extracted and displayed in the Arterial graphs and indicators panel (See section 4.6, Table 7), as well as related indicators.

If needed, compute transit time between two previously done ROIs by selecting both ROI colors in the Flow delay estimation panel.

Indices	Units	Descriptions
Max Area	Square centimeters (cm ²)	Maximal area of segmented section during the cardiac cycle
Min Area	Square centimeters (cm ²)	Minimal area of segmented section during the cardiac cycle
Variation	Percent (%)	Ratio of max-min area over min area. Also referred as Strain
Mean Diameter	Millimeters (mm)	Average diameter of the segmented section during the cardiac cycle (computed from area with a circular hypothesis)
Min Diameter	Millimeters (mm)	Minimal diameter of the segmented section during the cardiac cycle (computed from area with a circular hypothesis)
Max Diameter	Millimeters (mm)	Maximal diameter of the segmented section during the cardiac cycle (computed from area with a circular hypothesis)

Indices	Units	Descriptions
Diameter Variation	Percent (%)	Ratio of max-min diameter over min diameter
Beginning of ejection	Milliseconds (ms)	Time instant where the cardiac ejection begins
Ejection peak	Milliseconds (ms)	Time instant where the cardiac ejection flow peak occurs
End of ejection	Milliseconds (ms)	Time instant where the cardiac ejection terminates
End of regurgitation	Milliseconds (ms)	Time instant where the cardiac flow regurgitation terminates
Acceleration volume	Milliliters (mL)	Blood volume sent by the heart from the beginning of ejection to the ejection peak
Ejection volume	Milliliters (mL)	Blood volume sent by the heart during the total ejection phase
Regurgitation volume	Milliliters (mL)	Blood volume going backward due to regurgitation
Net volume	Milliliters (mL)	Blood volume flowing through during the whole cardiac cycle
Regurgitation fraction	Percent (%)	Ratio of the regurgitation volume over the net volume.
Flow at ejection peak	Milliliters per seconds (mL/s)	Flow value recorded at the ejection peak
Ejection acceleration slope	Milliliters per seconds per milliseconds (mL/s/ms)	First derivative (or slope) of the flow curve between the ejection start and peak. Quantifies the acceleration of flow.
Ejection start positive/negative	Milliseconds (ms)	Time instant where the cardiac ejection begins considering positive/negative flow only
Ejection peak positive/negative	Milliseconds (ms)	Time instant where the cardiac ejection flow peak occurs considering positive/negative flow only
Ejection end positive/negative	Milliseconds (ms)	Time instant where the cardiac ejection ends considering positive/negative flow only
Ejection volume positive/negative	Milliliters (mL)	Blood volume flowing through during the total ejection phase considering positive/negative flow only
Net volume positive/negative	Milliliters (mL)	Blood volume flowing through during the whole cardiac cycle considering positive/negative flow only
Flow at ejection peak positive/negative	Milliliters per seconds (mL/s)	Flow value recorded at the ejection peak for positive/negative flow
Transit Time (Require 2 arterial section ROIs)	Milliseconds (ms)	Time delay for the pulse wave to go from first arterial section to second one

Table 9 – Cine PC Indices description

7.1.2 BACKGROUND VELOCITY CORRECTION (CINE PC SEQUENCE ONLY)

This tool is used to rectify the velocity values by subtracting the background noise.

1. Click the Background Correction button
2. Design a circular area that represents a non-moving physical part and velocities are artifacts, by two clicks, one for the circle center and one for the circle ridge.

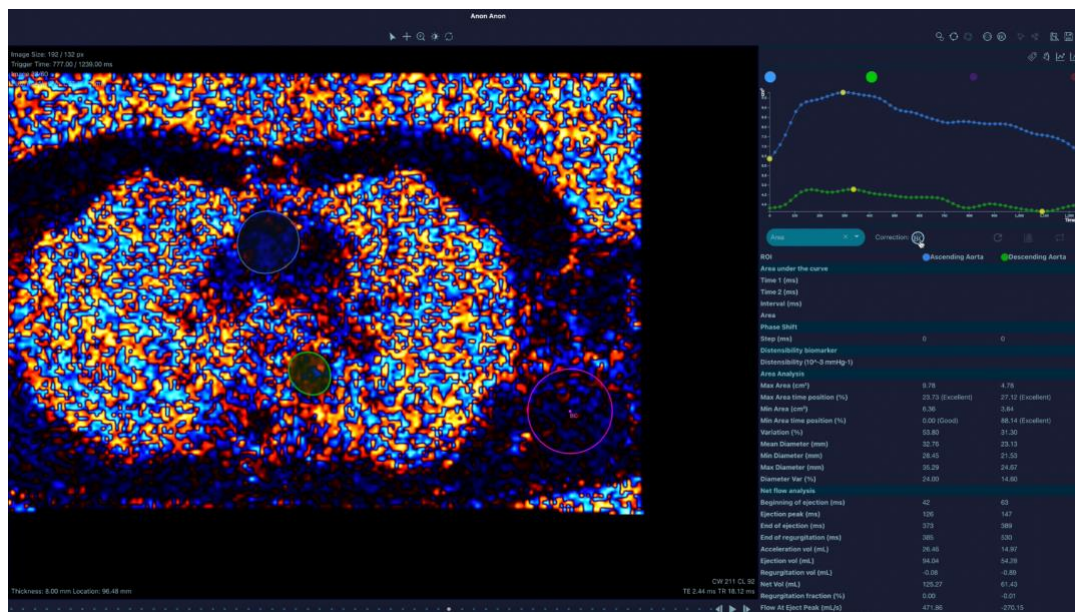
Figure 24 - Background Velocity correction

Velocity values are automatically recomputed.

7.1.3 CINE (2D+T) SSFP

The area curve is automatically extracted and displayed in the Arterial graphs and indicators panel, as well as related indicators.

Indices	Units	Descriptions
Max Area	Square centimeters (cm²)	Maximal area of segmented section during the cardiac cycle
Min Area	Square centimeters (cm²)	Minimal area of segmented section during the cardiac cycle
Variation	Percent (%)	Ratio of max-min area over min area. Also referred as Strain
Mean Diameter	Millimeters (mm)	Average diameter of the segmented section during the cardiac cycle (computed from area with a circular hypothesis)
Min Diameter	Millimeters (mm)	Minimal diameter of the segmented section during the cardiac cycle (computed from area with a circular hypothesis)
Max Diameter	Millimeters (mm)	Maximal diameter of the segmented section during the cardiac cycle (computed from area with a circular hypothesis)
Diameter Var	Percent (%)	Ratio of max-min diameter over min diameter

Table 10 – Cine SSFP indices descriptions

7.2 CSV EXPORT

ArtFun+ allows the results of the session to be saved locally on the current computer. The “Export CSV” button from the Workflow panel of the interface (See above section 4.5, Table 6) displays a window where the user can indicate their estimation of the image quality, their confidence in the measurements, and any useful comments (see figure below).

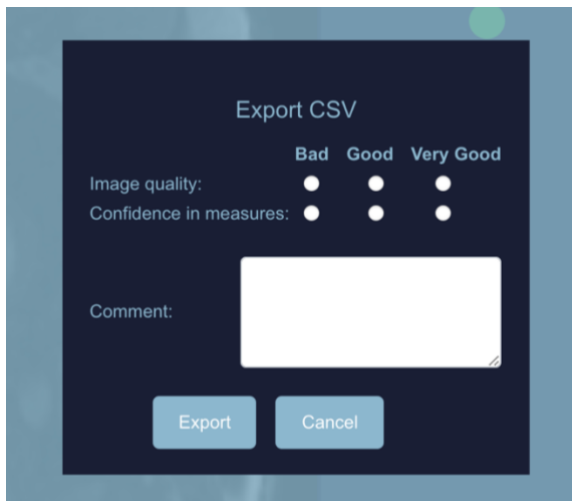
Figure 21 - Export CSV window

That information is not mandatory and is recorded for information only and has no impact on the generated files or results.

Click on the “Export” button to save the results. It generates 6 files located in the "ImageensReport" folder on the desktop:

- ArtFunPlus_Records_VascularResults, containing all clinicals indicators from cine arterial ROIs sorted by record date. It includes AAD & PWV results.
- ArtFunPlus_Records_Curves, containing all the curves values from cine arterial ROIs sorted by record date
- ArtFunPlus_Records_ROIs, containing all the coordinates of ROIs in image and patient space from cine arterial ROIs, sorted by record date
- ArtFunPlus_Records_VelocityPixels, containing velocity values for pixels inside segmentation
- Geometry_Records_LengthResults, containing the arterial length from 3D arterial ROIs sorted by record date
- Geometry_Records_ROIs, containing all the coordinates of ROIs in image and patient space from 3D arterial ROIs, sorted by record date.

All records can be identified (inside each csv file) by the timestamp (date & time) but also by patient and image information:

- Patient ID
- Patient Name
- Patient Age
- Patient Weight

- Study instance UID
- Series instance UID
- Series Description
- Image Position Patient (image origin in patient space)
- Image Orientation Patient (image orientation in patient space)
- Image Size
- Pixel Spacing
- ITK origin and scaled direction (A+ internal reference system)
- Sequence Type

WARNING: The csv file used for export must be closed and not used by any other applications during the use of ArtFun+ to prevent corrupting the file.

8 ARTFUN+ POP-UPS

There are different types of pop-up windows in ArtFun+. Pop-ups can appear either for information or to indicate that an action is required. These pop-ups are shown below.

8.1 INFORMATIVE POP-UPS

All the pop-up windows shown below are for information purposes only. No action is required to proceed, however, some pop-ups inform you of an error. It is necessary to take this information into consideration in order to obtain a correct result at the end of the analysis. The steps to be taken to resolve each error pop-up are listed in order of priority below each pop-up below.

1. Loading image in progress
2. Loading error
 - a. Check that the image sequence type is supported by ArtFun+. (Cine SSFP or Cine PC or 3D SSFP)
 - b. Check that the data are not corrupted or incomplete
 - c. Ensure that DICOMs are unzipped
 - d. If none of the above steps solves the error message you can contact IMAGEENS via the contacts listed in part 7 of this manual.
3. Error in the automatic creation of the initial circle
 - a. This error message will automatically generate the action pop-up "How to perform a segmentation correction manually".
4. Segmentation in progress
5. Segmentation error
 - a. Check that the target object is an artery
 - b. Check that the image does not contain any errors
 - c. Re-trying segmentation
 - d. If none of the above steps solves the error message you can contact IMAGEENS via the contacts listed in part 7 of this manual.

6. Region of interest analysis error
 - a. Ensure that the data is consistent and uncorrupted, e.g. for a PC sequence, check that there is the same number for morphology and phase images)
 - b. If the above step does not solve the error message you can contact IMAGEENS via the contacts listed in part 7 of this manual.
7. CSV export successful

8.2 ACTION POP-UPS

The pop-ups requiring an action are divided into two categories according to the type of action to be performed. Some pop-ups will ask you to perform an action on the ArtFun+ plug-in window, others will ask you to interact directly with the pop-up.

Pop-up involving an action on ArtFun+:

1. How to make the bounding box for segmentation
2. How to make the initial circle manually
3. How to perform a segmentation correction manually
4. How to position the points for modeling the central aortic line
5. How to correct the points for modeling the central aortic line

Pop-up involving an action on the pop-up:

1. Request for automatic initial circle validation
2. Selection of the first frame for semi-automatic segmentation correction
3. Selection of the last frame for semi-automatic segmentation correction

Pop-ups 2 and 3 of this category ask you to select the first and last frame where you want to make a segmentation correction. ArtFun+ will then automatically correct the segmentation of the aorta between these frames.

9 CLOSE ARTFUN+

To close ArtFun+, you must click on the cross at the top left of the user interface window.

WARNING: Make sure you save your work before closing the program. Any unsaved data will be lost when you close the program.

10 ARTFUN+ UPDATES

The update of ArtFun+ must be done by the user. An email will be sent to the user when an update is available. This email will contain a link to download the new version of the product.

The installation of this new version must be done in the same way as below:

1. Ensure the previously mentioned version of Horos OR Osirix is installed.

2. Unzip and launch the installer file by double click.
3. Osirix / Horos will start and ask you if you want to install the plugin. Osirix / Horos will then ask you to overwrite the previous version of the plug-in, answer yes.
4. Osirix or Horos will ask you to restart to complete the installation.
5. After a restart of Osirix/Horos, you will find the plugin in the menu “Plugins” ► “Image Filters”.
6. In the Osirix/Horos 2D Viewer panel, go in the menu “Format” ► “Customize Toolbar and drag & drop ArtFun+ into the usual toolbar to create a shortcut to ArtFun+.

11 UNINSTALL ARTFUN+

You cannot uninstall ArtFun+ directly from the program. If you want to do so, you need to remove the application from your computer system directly.

Note: If you reinstall ArtFun+ and if the license is still valid, the program will recognize it automatically.

12 ARTFUN+ PERFORMANCES & CLINICAL BENEFITS

PERFORMANCES

The technical performance for ArtFun+ is defined as its ability to produce accurate measurements, i.e. segmentations and related biomarkers, compared to a validated standard, in a reproducible and robust manner.

Dataset

IMAGEENS used a dataset of 100 Cardiac MRI in DICOM format to evaluate ArtFun+ performances. The size of the dataset is comparable to datasets used for evaluation of comparators in the state of the art and covers all the categories usable by ArtFun+: SSFP, PC and 3D SSFP sequences.

Method

The Ground Truth (GT) has been realized by performing the segmentations of:

- the Ascending Aorta (AA) section on cine SSFP sequence,
- the Ascending Aorta (AA) and Descending Aorta (DA) sections on cine PC sequence
- the Aortic Centerline on 3D SSFP sequence,

on each case of the specified dataset by a medical imaging expert.

The same segmentations are done on each case of the specified dataset by 2 different testers trained to medical image reading through the ArtFun+ software (Tested).

Both results are analyzed, separately between SSFP and PC sequences, according to previously set standards from scientific publications:

- Dice coefficient between GT and Tested segmentations for delineation correctness
- Intraclass correlation (ICC) on aortic parameters for inter-tester reproducibility
- ICC on aortic parameters for intra-tester reproducibility

Summary results

Sequences		SSFP		PC	
Performances	Score	Mean	[IC 0.95]	Mean	[IC 0.95]
Aorta segmentation	DICE	0.995	[0.994 – 0.996]	0.984	[0.980 – 0.987]
	Worst DICE	0.98	[0.977 – 0.983]	0.914	[0.899 – 0.928]
Intra-observer reproducibility	ICC	0.996	[0.994 – 0.997]	0.983	[0.968 – 0.999]
Inter-observer reproducibility	ICC	0.994	[0.990 – 0.998]	0.978	[0.947 – 1.000]

Discussion

DICE's score for both SSFP and PC sequences is higher than the claimed score of 0.84. DICE's average worst score is also higher than the claimed rate.

Regarding inter-observer reproducibility, for both SSFP and PC sequences, the ICC is higher than claimed, i.e., 0.97

Regarding intra-observer reproducibility, for SSFP sequences, the ICC is higher than claimed, i.e., 0.99 but for PC sequences the reproducibility is a bit lower (i.e. 0.983) than claimed but given that the confidence interval for this criterion is [0.98, 0.99], this is deemed acceptable.

The performance of ArtFun+ has been demonstrated to be consistent with its intended use.

CLINICAL BENEFIT

ArtFun+ Clinical Benefit is defined as the ability to handle patient management is a faster and more reliable fashion.

Dataset

The evaluation has been done on 5 MRI sequences.

Method

The objective of the evaluation is to compare the speed and the reproducibility of vessel segmentation on the dataset between the ArtFun+ method and a manual method.

As in the performance evaluation, the reproducibility has been evaluated by the ICC criterion from repeated Artfun+ measurements of the same structure: twice by the same tester (intra-tester reproducibility) and by two testers (inter-tester reproducibility).

Summary results & discussions

- ArtFun+ greatly accelerate the obtention of biomarker: 35 times faster in average, which is a direct positive impact on patient management, freeing time to perform more patients analyzes in the same period.

- Both reproducibility experiments demonstrate that ArtFun+ allows to mitigate operator-dependent bias: the reproducibility between two operators, improved by 24.8%, is in the same order as the reproducibility between two analyses of the same operator (which is improved by 5.6%), and in both case is excellent. Patient management is here also improved: any (trained) available operator can take on a patient case to improve workflow output.

13 USER DOCUMENTATION & SUPPORT

13.1 HELP MENU

Clicking on the “Help” tab on the menu bar opens up a drop-down menu (Figure 22 below) comprised of the following items:

- Contact us: when clicking on this item, you will be redirecting to Imageens website.
- Product Label: When clicking on this item, ArtFun+ Label will be displayed
- Instruction or use: When clicking on this item, ArtFun+ instruction for use will be displayed
- Learn More : When clicking on this item, you will be redirecting to Imageens website.

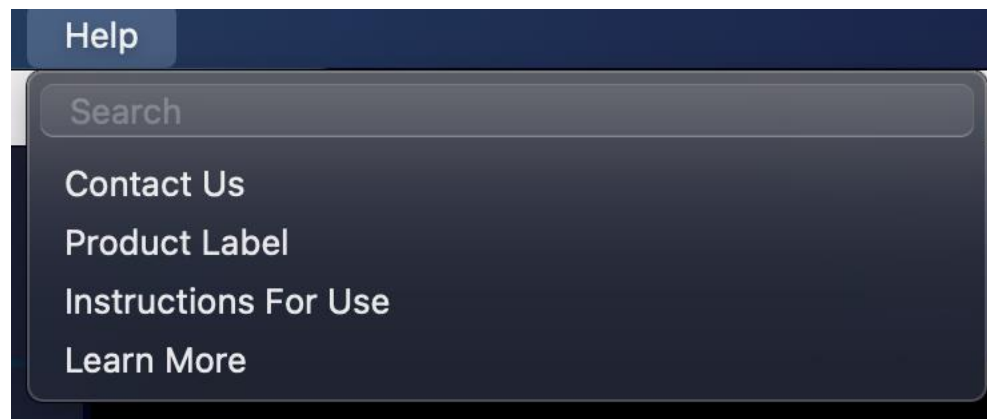
Figure 22 - ArtFun+ help menu

13.2 SUPPORT

Do you have trouble using the plugins? Have you encountered a bug?

Mail us at support@imageens.com and we will help you troubleshoot your issues!

A paper copy of the present IFU can be sent to you upon request by writing at the email above. It will be sent to you within 7 days from the request reception at no cost.

14 SYMBOLS

Symbol	Name	Description
--------	------	-------------

	Manufacturer	This symbol appears on the product label. It is accompanied by the manufacturer's name & address
	Date of manufacture	This symbol appears on the product label. It is accompanied by the product version release date (date of manufacture).
	Catalogue Number	This symbol displayed on the label is accompanied by the manufacturer's catalogue number.
	Consult electronic Instructions for Use	This symbol displayed on the label indicates that the Instructions for Use for the device are in electronic form and should be consulted.
	CE mark	This symbol displayed on the label indicates that the medical device has been awarded the CE mark by the notified body BSI (2797).
	Medical Device	This symbol displayed on the label indicates that ArtFun+ is a medical device
	UDI	This symbol displayed on the label indicates the Unique Device Identifier for ArtFun+ Release

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The image shows a 'Software Activation' dialog box with a dark blue background. It contains the following elements: a title 'Software Activation', a question 'You don't have a key ?', an email input field with 'example@email.com', a 'Request activation' button, a confirmation message 'We will send you an email as soon as possible', a 'License Key' label, a large text input field for the license key, and an 'Activate License' button.

Figure 6 - Activation d'ArtFun+

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15 REFERENCES

- Benetos A. Pulse pressure and cardiovascular risk. J Hypertens Suppl. 1999 Dec;17(5):S21-4. PMID: 10706321.
- Nichols WW, Edwards DG. Arterial elastance and wave reflection augmentation of systolic blood pressure: deleterious effects and implications for therapy. J Cardiovasc Pharmacol Ther. 2001 Jan;6(1):5-21. doi: 10.1177/107424840100600102. PMID: 11452332.
- Nussbacher A, Gerstenblith G, O'Connor FC, Becker LC, Kass DA, Schulman SP, Fleg JL, Lakatta EG. Hemodynamic effects of unloading the old heart. Am J Physiol. 1999 Nov;277(5):H1863-71. doi: 10.1152/ajpheart.1999.277.5.H1863. PMID: 10564141.
- Franklin SS, Khan SA, Wong ND, Larson MG, Levy D. Is pulse pressure useful in predicting risk for coronary heart Disease? The Framingham heart study. Circulation. 1999 Jul 27;100(4):354-60. doi: 10.1161/01.cir.100.4.354. PMID: 10421594.
- Ohtsuka S, Kakihana M, Watanabe H, Sugishita Y. Chronically decreased aortic distensibility causes deterioration of coronary perfusion during increased left ventricular contraction. J Am Coll Cardiol. 1994 Nov 1;24(5):1406-14. doi: 10.1016/0735-1097(94)90127-9. PMID: 7930267.
- Blacher J, Safar ME, Guerin AP, Pannier B, Marchais SJ, London GM. Aortic pulse wave velocity index and mortality in end-stage renal disease. Kidney Int. 2003 May;63(5):1852-60. doi: 10.1046/j.1523-1755.2003.00932.x. PMID: 12675863.
- Boutouyrie P, Tropeano AI, Asmar R, Gautier I, Benetos A, Lacolley P, Laurent S. Aortic stiffness is an independent predictor of primary coronary events in hypertensive patients: a longitudinal study. Hypertension. 2002 Jan;39(1):10-5. doi: 10.1161/hy0102.099031. PMID: 11799071.

8. Laurent S, Boutouyrie P, Asmar R, Gautier I, Laloux B, Guize L, Ducimetiere P, Benetos A. Aortic stiffness is an independent predictor of all-cause and cardiovascular mortality in hypertensive patients. *Hypertension*. 2001 May;37(5):1236-41. doi: 10.1161/01.hyp.37.5.1236. PMID: 11358934.
9. Meaume S, Benetos A, Henry OF, Rudnichi A, Safar ME. Aortic pulse wave velocity predicts cardiovascular mortality in subjects >70 years of age. *Arterioscler Thromb Vasc Biol*. 2001 Dec;21(12):2046-50. doi: 10.1161/hq1201.100226. PMID: 11742883.
10. Safar ME, Henry O, Meaume S. Aortic pulse wave velocity: an independent marker of cardiovascular risk. *Am J Geriatr Cardiol*. 2002 Sep-Oct;11(5):295-8. doi: 10.1111/j.1076-7460.2002.00695.x. PMID: 12214167.
11. Laurent S, Cockcroft J, Van Bortel L, Boutouyrie P, Giannattasio C, Hayoz D, Pannier B, Vlachopoulos C, Wilkinson I, Struijker-Boudier H; European Network for Non-invasive Investigation of Large Arteries. Expert consensus document on arterial stiffness: methodological issues and clinical applications. *Eur Heart J*. 2006 Nov;27(21):2588-605. doi: 10.1093/eurheartj/ehl254. Epub 2006 Sep 25. PMID: 17000623.
12. McDonald DA. Regional pulse-wave velocity in the arterial tree. *J Appl Physiol*. 1968 Jan;24(1):73-8. doi: 10.1152/jappl.1968.24.1.73. PMID: 5635773.
13. Groenink M, de Roos A, Mulder BJ, Spaan JA, van der Wall EE. Changes in aortic distensibility and pulse wave velocity assessed with magnetic resonance imaging following beta-blocker therapy in the Marfan syndrome. *Am J Cardiol*. 1998 Jul 15;82(2):203-8. doi: 10.1016/s0002-9149(98)00315-4. PMID: 9678292.
14. Laffon E, Marthan R, Montaudon M, Latrabe V, Laurent F, Ducassou D. Feasibility of aortic pulse pressure and pressure wave velocity MRI measurement in young adults. *J Magn Reson Imaging*. 2005 Jan;21(1):53-8. doi: 10.1002/jmri.20227. PMID: 15611948.
15. Mohiaddin RH, Firmin DN, Longmore DB. Age-related changes of human aortic flow wave velocity measured noninvasively by magnetic resonance imaging. *J Appl Physiol* (1985). 1993 Jan;74(1):492-7. doi: 10.1152/jappl.1993.74.1.492. PMID: 8444734.
16. Vulliémoz S, Stergiopulos N, Meuli R. Estimation of local aortic elastic properties with MRI. *Magn Reson Med*. 2002 Apr;47(4):649-54. doi: 10.1002/mrm.10100. PMID: 11948725